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
Introduction / To keep in mind.

For the best performance and weighing results, take care in the daily use and maintenance of your scale.

For long durability and best results ensure that:

- DO NOT overload the scale. This will damage the load cell and void the warranty.
- DO NOT leave the scale with a load on it for an extended period. This reduces the accuracy of the scale and shortens the life of the load cell.
- DO NOT try to repair the scale yourself, in case of problems, contact your local representative.
- Avoid exposing the scale to temperature differences. Do not place the scale in direct sunlight or near cooling equipment.
- Make sure that the scale is on a stable surface and that it is not subject to vibrations. If your scale is equipped with adjustable feet and spirit levels. Adjust if necessary.
- Do not place the scale near large electrical appliances, such as welders or large motors. Do not leave batteries in the scale if it is not in use for a long time.
- Avoid placing the scale in a humid environment where condensation can occur. Avoid exposing the scale to direct contact with water.
- DO NOT place the scale near windows, doors, or fans where drafts can affect the weighing result.
- Keep the scale clean. Wipe, if necessary, with a damp cloth.
- Be sure to use the original adapter/charger and battery if the scale is equipped with it.

Use

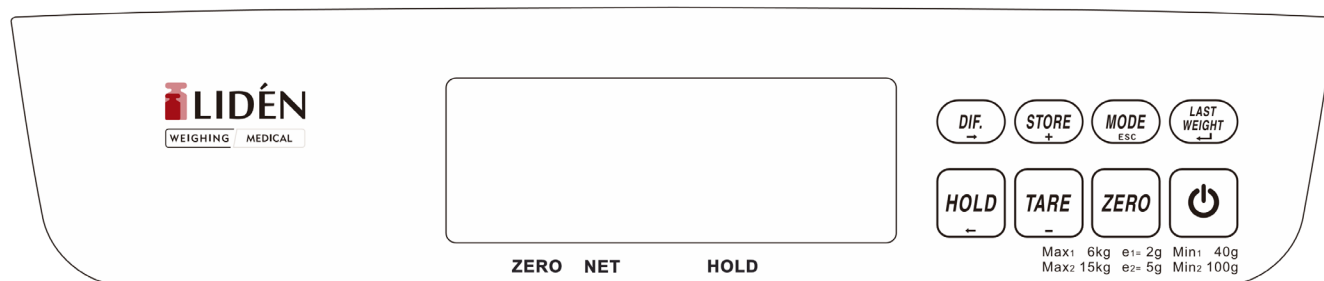Power On/Off

Press briefly to switch the scale on or off.

ZERO

Zero key

ZERO

Zero indication

Resets the scale 's display. Used for unloaded scale.

Press this button to reset the scale to zero, max weight. ($\pm 2\%$)

TARE

Tare key

NET

Net indication / Scale is tared in position.

The tare button is used to subtract things that are weighed with the child. How to: place the item on the scale. Press TARE to remove this weight, remove the object. The display will show NET. Carry out the weighing of the child. Press ZERO to exit the tare function.

Preset tare

Hold down TARE for 3 seconds. Enter the known tare weight with the arrow and +/- keys and confirm with LAST WEIGHT.

Carry out the weighing and then remove the child.

The display shows a negative value. Press the ZERO button to exit the tare function.

Hold key

HOLD

Hold indication

When the child moves, the weight in the display also moves. You can freeze the weight with the HOLD button.

You can switch between automatic hold function or manual by pressing down the HOLD button for 3 seconds.

- Automatic Hold (ON): The scale locks the displayed value automatically.
- Manual Hold (OFF): Press the HOLD button, the scale locks the weight value.

STORE
+

Storage of weight value

Place the child on the scale, when the scale is stable, press STORE+ for 3 seconds. Display indicates STO 00. Use STORE+ and TARE to enter memory position (0-9), save with LAST WEIGHT.

Press MODE to return to weighing function or wait 10 seconds for automatic return.

Use ZERO to reset memory position.

Press MODE to return to weighing mode.

DIF.

Differential weighing

Used to be able to weigh, for example, how much a child has eaten by weighing before and after with a stored weight value.

Place the child on the scale. When the scale is stable, press the DIF button.

Use +/- to enter memory position (AR 0-9). Press LAST WEIGHT to store the weight. Lift up and feed the child.

Place the child back on the scale and press DIF and then LAST WEIGHT to display the current weight, stored weight, and the difference.

The scale returns to normal after 3 seconds.

MODE Mode and exit button
ESC

By pressing MODE, you can choose between two measurement ranges:
1=0-6kg and 2=0-15 kg.

Select Auto and both ranges 0–6 kg/2g and 6–15 kg/5g are used.

LAST Last weight /value

WEIGHT For the function for displaying the latest weight/value to work, the
automatic hold function must be activated (Automatic Hold (ON),
see section Hold function.

Background lighting off: Press LAST WEIGHT for 3 seconds and the
display will show OFF.

Background lighting on: Press LAST WEIGHT for 3 seconds and the
display will show ON.

The background lighting is, in this mode, on when scale is actively used.

o Stable value

Measuring range:

The scale has 2 measuring ranges 1=0-6Kg and 2=0-15 Kg You can select this with
the MODE button

Select Auto and you have both ranges 0-6 kg 2g and 6-15 kg 5g (no indication on the
display)

Example of use:

Switch on the scale by briefly pressing the on/off button.

Make sure the display shows 0, if not, reset with the >0< button.

Put on the blanket and press the Tare button.

Place the child in the weighing pan and press the HOLD button, wait until the arrow
lights up at HOLD.

The weight can be displayed again by pressing the LAST WEIGHT button.

This value disappears if you turn off the scale.

Maintenance


Battery

Type: 1,5V, alkaline

Quantity: 6

Replacing the batteries

1. Disconnect the power adapter to the scale.
2. Remove the battery cover on the back of the indicator.
3. Insert new AA batteries and make sure the polarities are inserted correctly.
4. Close the battery cover.

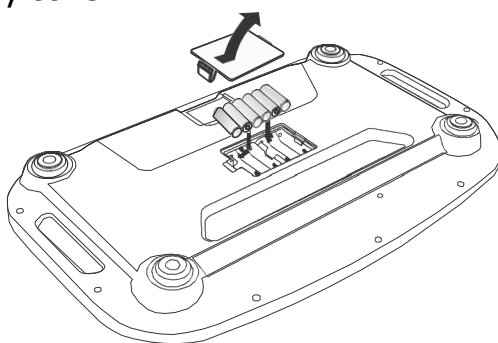NOTE! If you use rechargeable batteries, the batteries should be charged in an external charger. The scale adapter cannot charge the batteries when they are in the scale.

The scale is set to switch off after 5 minutes to save the batteries.

When the low battery symbol appears in the left corner of the display, the batteries must be replaced. If LOBAT appears in the display, the batteries must be replaced immediately.

Cleaning / Disinfection

For all maintenance with the scale, the power supply must be disconnected.

The scale may only be cleaned with household cleaners or disinfectants, e.g. 70% isopropanol solution.

We recommend a disinfectant that is intended for cleaning by damp cloth the surfaces of the scale. Follow the manufacturer's instructions.

Do not use polishing agents or strong cleaning agents such as petrol, as they can damage the plastic or rubber surfaces.

To avoid cross-contamination, the following disinfection occasions are recommended:

- Before and after any direct contact with the skin.
- Scale display - if required.
- Immediately remove all visible contamination.

Do not spray the disinfectant on the scale display (with pressure)

The disinfectant must not penetrate the scale.

Control routines

The scale is verified upon delivery and must be reverified if the seal (SEAL) is broken during, for example, repair of the scale, or if it is suspected that the scale is showing errors.

Own checks can be made with known weights but are not necessary.

Contact your dealer if questions arise.

Error codes

If the scale does not work as it should or displays error codes, please contact your dealer.