


User manual

BW-0520



www.liden.weighing.com

Front panel

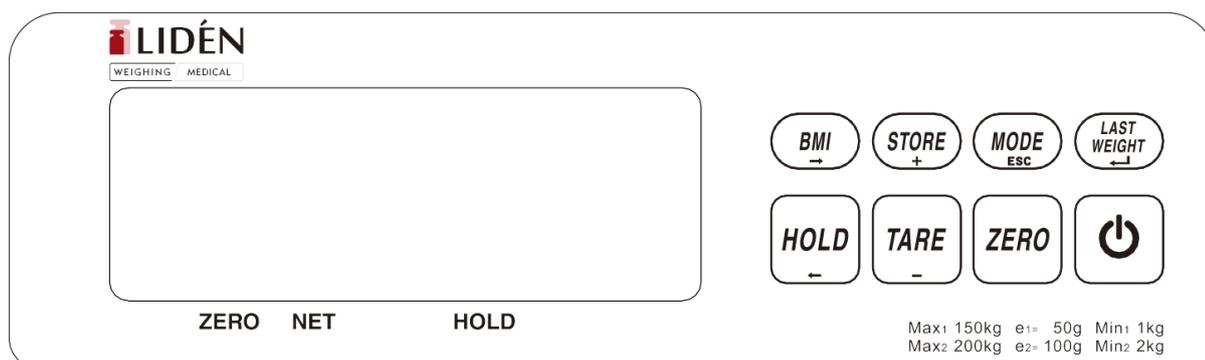

User manual

Backlight selection

1. No backlight: Press  lasting 3 seconds, it displays "OFF"
2. Auto backlight: Press  lasting 3 seconds, it displays "ON"
Backlight function will be active while loading.

2. Keypad function

Press to turn the scale on or off.



1. Power on: Press this key briefly
2. Power off: Press this key lasting 2 seconds and it will display OFF.



In ZERO mode, hold down this key to select AUTO ON (Auto Hold) or AUTO OFF (Manual Hold).

1. Auto Hold Mode:
The scale will auto hold after the weight is stable.
2. Manual Hold Mode:
After the weight is stable, press this key to hold.

3. Hold again after weighing lock:

The weight is stable and lock.  Press to release hold function and hold again.

TARE function (press briefly)



1. Place the empty container on the scale, press  key briefly to deduct the weight of the container. The display will show negative value after remove the empty container.
2. Press  or  to cancel this function.

Pre-tare function (Hold down  lasting 3 seconds)

1. In zero mode, hold down this key lasting 3 seconds. Key in the known weight and press  key. The screen displays the negative weight after Pre-taring. Press  or  to cancel this function.

2. While loading, press  key to display “pre-tare value” lasting 3 seconds.

1. Press this key to adjust zero range (<+/-2%)



2. When the slide switch in “ON” position and the screen shows ZERO, hold down this key after hearing a beep sound. It will display high resolution value.

3. Press  key and then turn on the power and you will see the software version. Release the keys and it will go back to weighing mode.



In ZERO mode, press this key briefly to enter multi-range mode |1| |2| auto



Store weight value:

Place baby on the platform and it will hold the baby’s weight after weight stable. Hold down this key lasting 3 seconds, it will display  and use  to key in the storage position(1~99)  (for example). Press  to finish.

Recall/clear the store memory:

In ZERO mode, press this key briefly and display  and use  to key in (desired memory position 1~99)  (foe example). Press  to select (1) or (2)

(1) Press  to leave or wait for 10 seconds to auto back weighing mode.

(2) Use  key to clear storage value. After clearing, it will display next memory position. You could go on to clear or press  to leave.



Body Mass Index:

The screen display “0”, ask the person step on the platform. When the weight is stable, press  key and use     to key in person’s height. Press  and it will show BMI automatically. The BMI value vanishes after the person step off the platform.




In ZERO mode, press this key briefly to display previous reading. Select (1) or (2)


Replace battery:

DO NOT USE rechargeable battery! If you do so, they must recharge in external charger. The main adapter cannot recharge the battery when they are in the scale.


1. Turn the scale over and loose two screws (1,2). Then move the indicator.

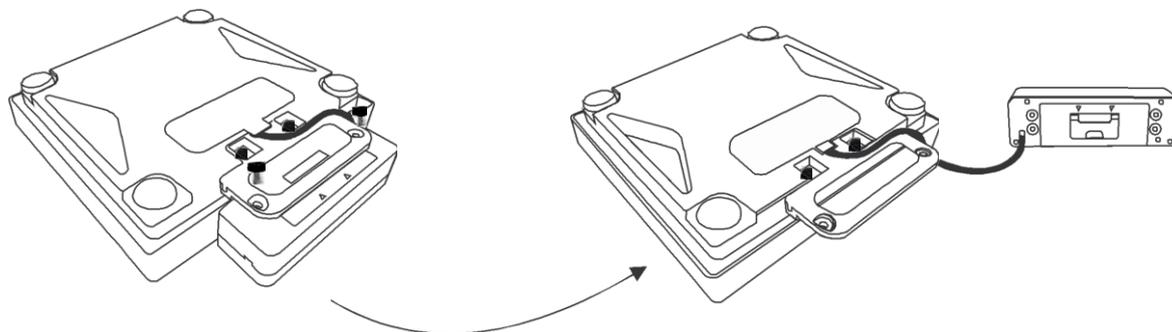

2. Open the battery cover as arrow.

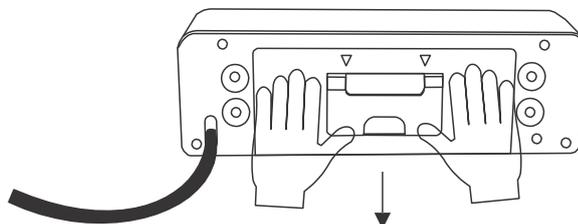

3. Insert batteries and make sure the polarity are correct.

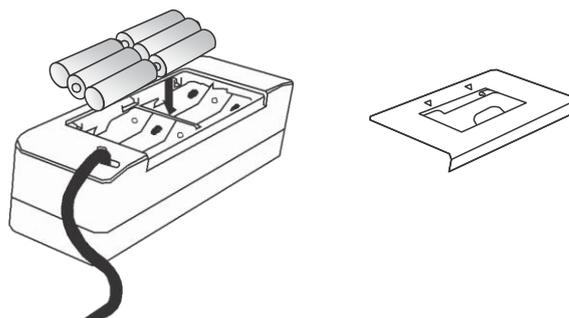

4. Close the battery cover.

Cleaning:

Before cleaning, make sure the scale has been disconnected from the socket.


1. Please do not use large amounts of water when cleaning the scales as this will cause damage to the scales electronics.
2. Don't use mild soap and water solution.
3. Clean all external surfaces with a clean damp cloth.

Maintenance and service:

If you have any queries or problems, please contact your local dealer. Don't try to repair it by yourself once the scale is damage or need re-calibration.


How to separate the indicator

1. Turn the scale over and loose two screws(1,2)

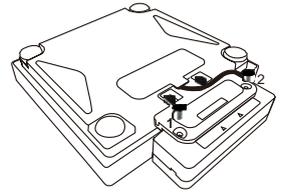

2. Open the signal cable box cover as arrow.

3. You could place the indicator in the desk or wall-mounted.

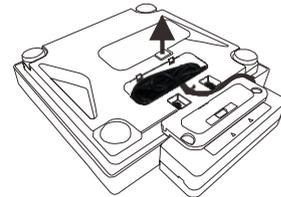

4. Close the cover of the signal cable box.



