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CMC/SRN-nummer: ES-AR-000000293


Introduction / To keep in mind.

For the best performance and weighing results, take care in the daily use and maintenance of your scale.


For long durability and best results ensure that:

- DO NOT overload the scale. This will damage the load cell and void the warranty.
- DO NOT leave the scale with a load on it for an extended period. This reduces the accuracy of the scale and shortens the life of the load cell.
- DO NOT try to repair the scale yourself, in case of problems, contact your local representative.
- Avoid exposing the scale to temperature differences. Do not place the scale in direct sunlight or near cooling equipment.
- Make sure that the scale is on a stable surface and that it is not subject to vibrations. If your scale is equipped with adjustable feet and spirit levels. Adjust if necessary.
- Do not place the scale near large electrical appliances, such as welders or large motors. Do not leave batteries in the scale if it is not in use for a long time.
- Avoid placing the scale in a humid environment where condensation can occur. Avoid exposing the scale to direct contact with water.
- DO NOT place the scale near windows, doors, or fans where drafts can affect the weighing result.
- Keep the scale clean. Wipe, if necessary, with a damp cloth.
- Be sure to use the original adapter/charger and battery if the scale is equipped with it.

Symbols on product and box

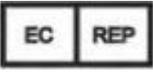	Caution, consult accompanying documents before use.		Separate collection for waste of electrical and electronic equipment, in accordance with Directive 2002/96/EC.
	Manufacturer of medical device.		Manufacturing year of medical device.
	Carefully read user manual before installation and usage and follow instructions for use.		Medical electrical equipment with Type B applied part.
	Device catalogue number.		Authorized representative in the European Community.
	Manufacturer's batch or lot number.		Device is a medical device.
	Serial number.		Unique Device Identifier.
		Device conforms to 93/42/EEC as amended by 2007/47/EC Medical Device Directive. Four-digit number refers to Notified Body.	
		Device complies with International Organization of Legal Metrology (Class III) requirements (verified models only).	
		<p>Device complies with EC directives (verified models only).</p> <p>M: Conformity label in compliance with Directive 2014/31/EU for non-automatic weighing instruments.</p> <p>18: Year in which conformity verification was performed and the CE label was applied. (ex: 18=2018).</p> <p>0122: Refers to Notified Body for metrology.</p>	

Use

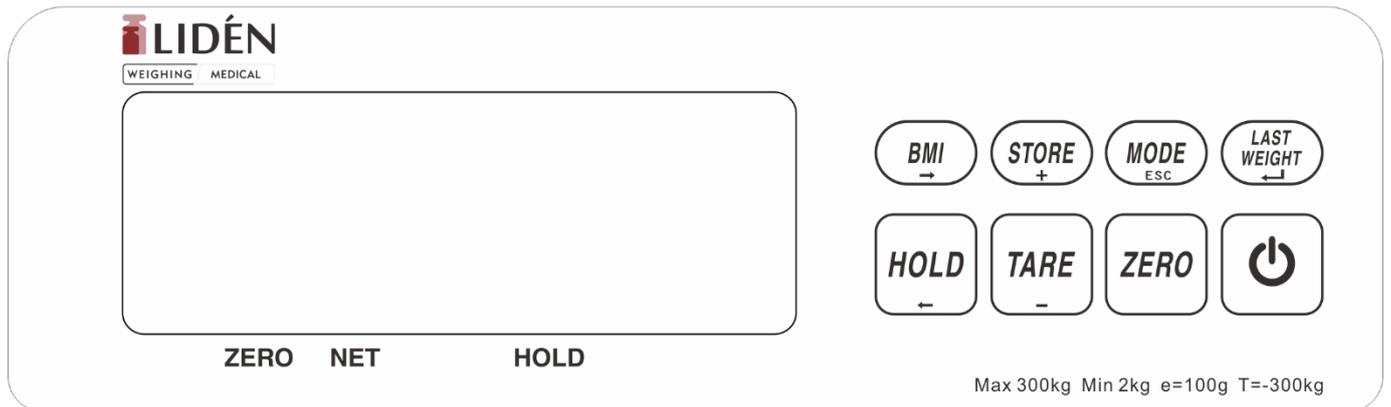
Power On/Off
Press briefly to switch the scale on or off.

Zero key

ZERO

Indication that the scale is within zero range.
Press this button to reset the scale. (<+/-2%).

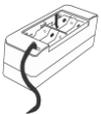Tare key

NET

Net indication / Scale is tared in position.

Press TARE with item on weighing platform to tare display. Remove item and press TARE again to exit the tare function.

Preset tare

Hold down TARE for 3 seconds. Enter the known tare weight with the arrow and +/- keys and confirm with LAST WEIGHT. Carry out the weighing of the person. When the person leaves the scale the display shows a negative value. Press TARE or ZERO to reset the function.

Hold key

HOLD

Hold indication

You can switch between automatic holdfunction or manual by pressing down the HOLD button for 3 seconds.

- **Automatic Hold (ON):** The scale locks the displayed value automatically.
- **Manual Hold (OFF):** Press the HOLD button, the scale locks the weight value.

Storage of weight value

Place the person stand on the scale, when the scale is stable, press STORE for 3 seconds. Display indicates STO 00. Use STORE and TARE to enter memory position (0-9), save with LAST WEIGHT.

Press MODE to return to weighing function or wait 10 seconds for automatic return. Use ZERO to reset memory position.

Press MODE to return to weighing mode.

To display weight values in storage, press STORE, select the storage location number and press LAST WEIGHT.

BMI (Body Mass Index)

Have the person stand on the and wait for stable indication.

When the scale is stable press BMI.

Use BMI, STORE, HOLD and TARE to enter the persons height Press LAST WEIGHT to display BMI.

The BMI value will disappear when the person leaves the scale.

Press LAST WEIGHT to display last weight, height and BMI.

Mode and exit button

By pressing MODE, you can choose between two measurement ranges:
1=150/0.05kg and 2=200/0.1kg.

Select Auto and both areas are used with a maximum weight of 200kg.

Last weight /value

Background lighting off: Press LAST WEIGHT for 3 seconds and the display will show OFF.

Background lighting on: Press LAST WEIGHT for 3 seconds and the display will show ON.

The background lighting is, in this mode, on when scale is actively used.

- o Stable value

Maintenance


Battery

Type: 1,5V, alkaline


Quantity: 6

Replacing the batteries

1. Disconnect any power adapter from the scale.
2. Open the battery cover in the direction shown by the arrow.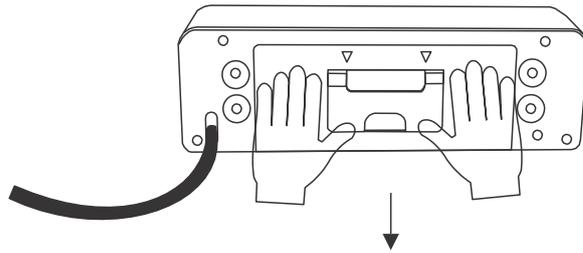

3. Insert new AA batteries and make sure the polarities are inserted correctly.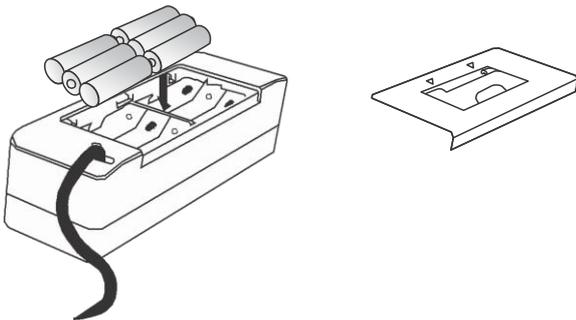

4. Close the battery cover.


NOTE! If you use rechargeable batteries, the batteries should be charged in an external charger. The scale adapter cannot charge the batteries when they are in the scale.

Using an adapter

1. Connect adapter to indicator before connecting to wall socket.
2. Disconnect adapter from wall socket before unplugging adapter from indicator.


Assembling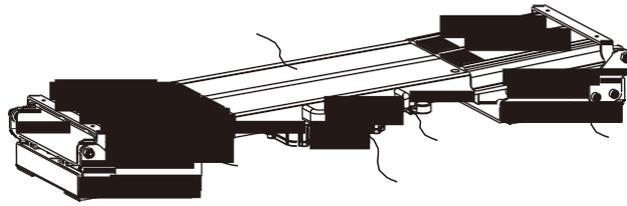


Ramp Handle/Junction box Bottom base


1. Turn the ramp outward.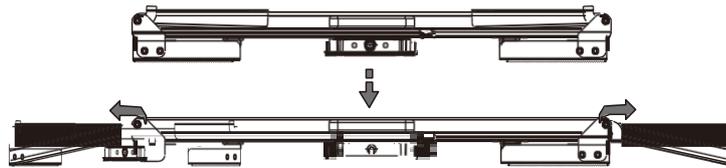


2. Connect 4 pin signal cable to junction box.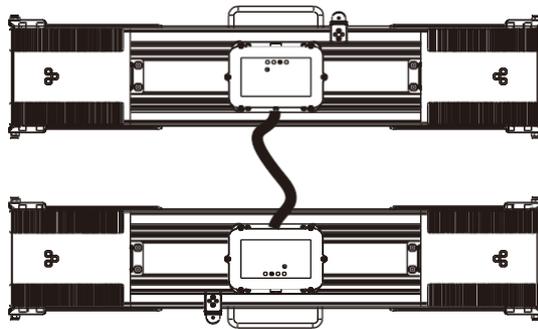

3. Connect 8 pin signal cable to the indicator.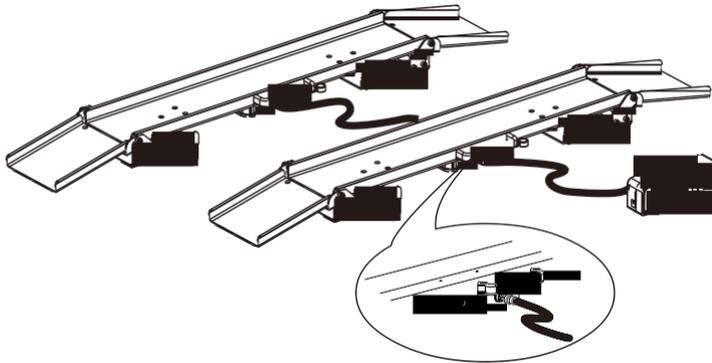

4. Make sure to leave a space between beam and ramp to avoid incorrect measuring readout.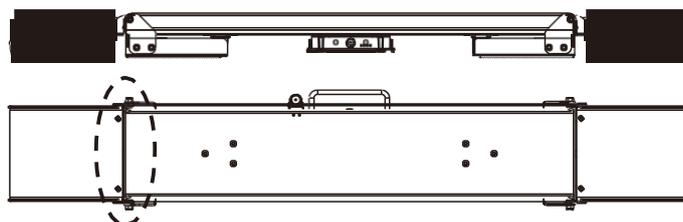

Note

When moving the scale, cables must be removed and reassembled.
Pay attention to how to reassemble the wiring.

Cleaning / Disinfection

For all maintenance with the scale, the power supply must be disconnected.

The scale may only be cleaned with household cleaners or disinfectants, e.g. 70% isopropanol solution.

We recommend a disinfectant that is intended for cleaning by damp cloth the surfaces of the scale. Follow the manufacturer's instructions.

Do not use polishing agents or strong cleaning agents such as petrol, as they can damage the plastic or rubber surfaces.

To avoid cross-contamination, the following disinfection occasions are recommended:

- Before and after any direct contact with the skin.
- Scale display - if required.
- Immediately remove all visible contamination.

Do not spray the disinfectant on the scale display (with pressure)

The disinfectant must not penetrate the scale.

Control routines

The scale is verified upon delivery and must be reverified if the seal (SEAL) is broken during, for example, repair of the scale, or if it is suspected that the scale is showing errors.

Own checks can be made with known weights but are not necessary.

Contact your dealer if questions arise.

Error codes

If the scale does not work as it should or displays error codes, please contact your dealer.

Intended use

Professional use in healthcare facilities. The device is intended to measure the weight of wheelchair users. Also, for personal use in home environment.

EMC Guidance and Manufacturer's Declaration

Guidance and manufacturer's declaration-electromagnetic emissions		
The MS4910 Stand-on Floor Scale is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment.		
Emission test	Compliance	Electromagnetic environment-guidance
RF emissions CISPR 11	Group 1	The device uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic environment. The device is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
RF emissions CISPR 11	Class B	
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations /flicker emissions IEC 61000-3-3	Compliance	

Guidance and manufacturer's declaration-electromagnetic immunity			
The MS4910 Stand-on Floor Scale is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment-guidance
Electrostatic discharge(ESD) IEC 61000-4-2	<u>± 8 kV contact</u> <u>± 2 kV, ± 4 kV,</u> <u>± 8 kV, ± 15 kV</u> <u>air</u>	<u>± 8 kV contact</u> <u>± 2 kV, ± 4 kV,</u> <u>± 8 kV, ± 15 kV</u> <u>air</u>	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%
Electrical fast transient/burst IEC 61000-4-4	± 2kV for power supply lines + 1kV for input/output lines	+ 2kV for power supply lines + 1kV for input/output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1kV line(s) to line(s) ± 2kV line(s) to earth	+ 1kV line(s) to line(s) + 2kV line(s) to earth	Mains power quality should be that of a typical commercial or hospital environment.
Voltage Dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<u>0% UT for 0,5 cycle</u> <u>0% UT for 1 cycle</u> <u>70% UT(30% dip in UT) for 25 cycles</u> <u>0% UT for 5 s</u>	<u>0% UT for 0,5 cycle</u> <u>0% UT for 1 cycle</u> <u>70% UT(30% dip in UT) for 25 cycles</u> <u>0% UT for 5 s</u>	Mains power quality should be that of a typical commercial or hospital environment. If the user of the device requires continued operation during power mains interruptions, it is recommended that the device be powered from an uninterruptible power supply or a battery.
Power frequency(50/60 Hz) magnetic field IEC 61000-4-8	<u>30 A/m</u>	<u>30 A/m</u>	The device power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE UT is the a.c. mains voltage prior to application of the test level.			

Guidance and manufacturer's declaration-electromagnetic immunity

The MS4910 Stand-on Floor Scale is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment-guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 KHz to 80 MHz	3 Vrms 150 KHz to 80 MHz	Portable and mobile RF communications equipment should be used no closer to any part of the device including cables, than the recommended separation distance calculated from the equation applicable to
Radiated RF IEC 61000-4-3	<u>6 V in ISM bands between 0,15 MHz and 80 MHz</u> 80 % AM at 1 kHz 3 V/m	<u>6 V in ISM bands between 0,15 MHz and 80 MHz</u> 80 % AM at 1	

	80MHz to 2,7 GHz	<u>kHz</u> 3 V/m <u>80MHz to 2,7 GHz</u>	<p>the frequency of the transmitter.</p> <p>Recommended separation distance: $d = 1,2 \sqrt{P}$ $d = 1,2 \sqrt{P}$ 80MHz to 800 MHz $d = 2,3 \sqrt{P}$ 800MHz to 2,5 GHz</p> <p>Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey^a, should be less than the compliance level in each frequency range^b.</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> <div align="center">  </div>
--	------------------	----------------------------------------------------	------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

NOTE1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

- a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the device is used exceeds the applicable RF compliance level above, the device should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the device.
- b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Recommended separation distance between portable and mobile RF communications equipment and the MS4910 Stand-on Floor Scale

The device is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the device as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = 1,2\sqrt{P}$	80 MHz to 800 MHz $d = 1,2\sqrt{P}$	800 MHz to 2,5 GHz $d = 2,3\sqrt{P}$
0,01	0,12	0,12	0,23
0,1	0,38	0,38	0,73
1	1,2	1,2	2,3
10	3,8	3,8	7,3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where p is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.