# Digital Personvåg

## MANUAL

H151-5 & H151-6

[www.liden-weighing.se](http://www.liden-weighing.se)

# Specifikationer

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
## Unpacking the Scale

Visually inspect the scale to ensure all components are included in the box. The box should contain the scale, this manual and a box labeled “parts inside”.

Packed scale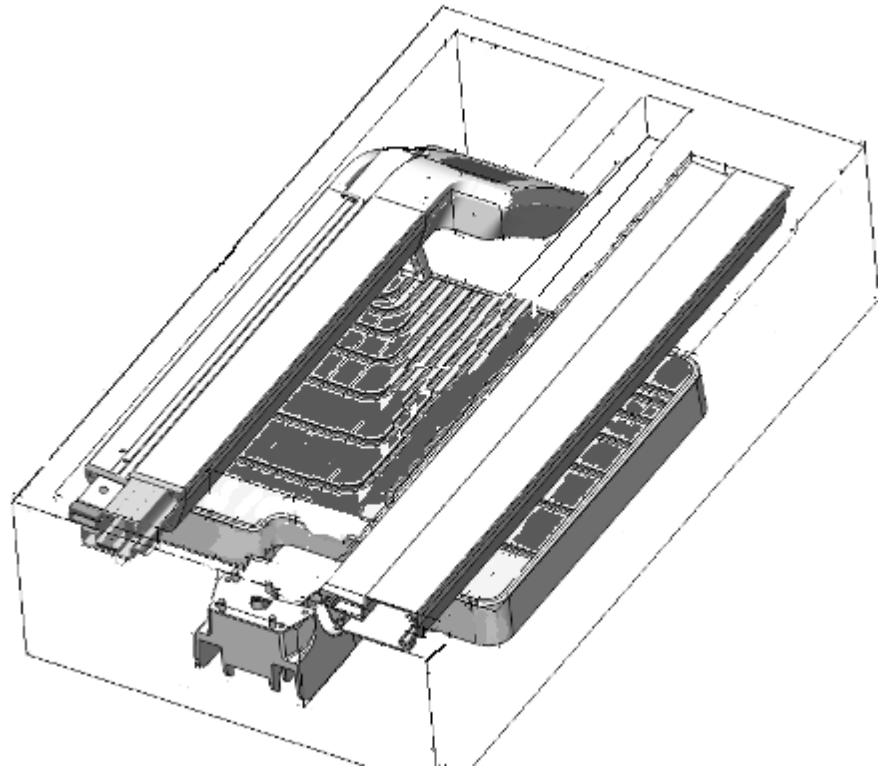

The “PARTS INSIDE” box includes the following parts:

- Two (2) Wheels.
- One (1) wheel hinge
- Five (5) pan head cross recess M5 x 35 screws.
- Two (2) pan head cross recess M4 x 16 screws.
- 6 AA batteries.
- RS232 communication cable 2.0 meter
- USB communication cable 2.0 meter (optional)


## Parts for Assembly

Remove the scale units from of the box, and arrange them on a clear work area

Scale platform

**Pay attention – This part is heavy, take care not to drop it.**

**Caution – Minor or Moderate injury may occur**

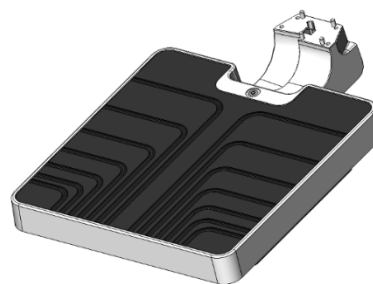
Upper pillar with Indicator

**Pay attention – mishandling this part may cause damage to the device or measured results**

Bottom pillar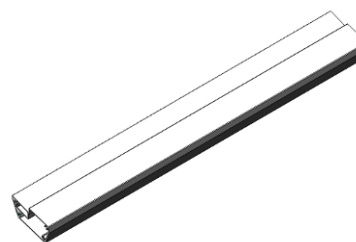

## Tools Required for Assembly:


Phillips screwdriver size 2.

## Assembling the Scale


Insert the bottom pillar to the scale platform, using the studs as guides

- Take care NOT to catch the load cell cable between these parts.
- Pay attention that there are two screw holes on the bottom pillar, which on assembly needs to be position upward.

Insert the four M5 screws and tighten them in place with Phillips screwdriver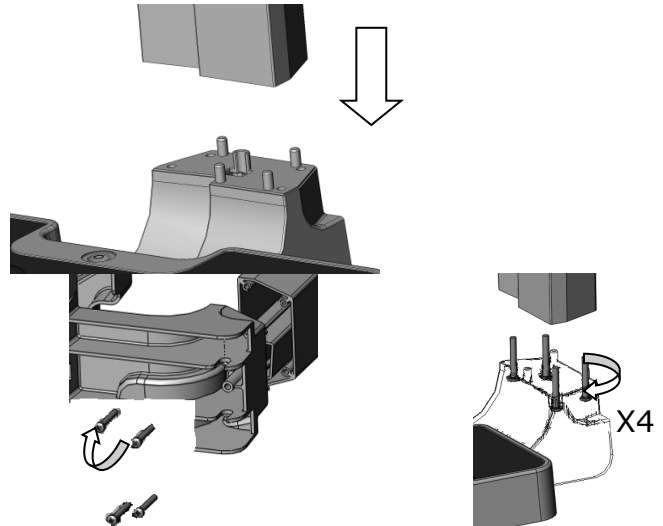

From the "PARTS INSIDE" box, take out the two wheels and the hinge. Place each wheel on the hinge and assemble them in the groove, locking in place using the M5 screw and a Phillips screw-driver.

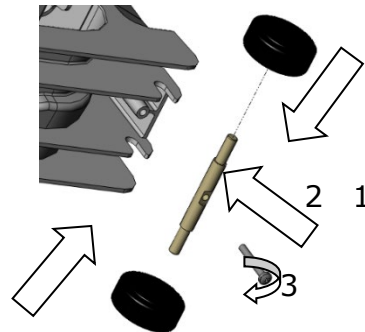Insert the upper pillar into the bottom pillar.

- **Take care not to damage the load cell cable. Insert the extra cable into the bottom pillar.**
- Insert the height rod edge into the appropriate rail
- 

Secure the upper unit assembly by screwing the two (2) M4 screws in the back of the unit.